


Bodyguard 2 guide for Mac OS X & Windows

Browser specific guide to install support for Firstbeat Bodyguard 2 to Sports Cloud.

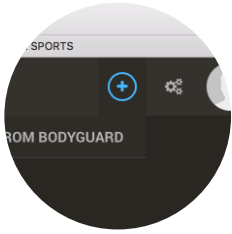

Select "Upload from Bodyguard" from Sports Cloud and then follow your browser specific guidance.

Safari Internet Explorer

1. Download & Install plugin
2. Restart browser

Measurements from Bodyguard 2 can be loaded using a browser plugin. Click the link below to download the plugin. After installation, restart your browser before using Bodyguard device.

[Download here](#)



CLOSE

Google Chrome

1. Add Extension to Chrome
2. Restart browser
3. Download & install plugin
4. Restart browser

To upload measurements from Bodyguard device you'll need to install a browser extension. Please install browser extension by clicking the button below. After installation please restart your browser.

[Add to Chrome](#)



CLOSE

Measurements from Bodyguard 2 can be loaded using a browser plugin. Click the link below to download the plugin. After installation, restart your browser before using Bodyguard device.

[Download here](#)



CLOSE


(In case you have Sports Monitor installed you may have plugin already installed.)

Mozilla Firefox

1. Download & install plugin
2. Restart browser
3. Enable web extension
4. Refresh browser

1. Measurements from Bodyguard 2 can be loaded using a browser plugin. Click the link below to download the plugin. After installation restart your browser and enable browser extension from browser's top right menu.

[Download here](#)

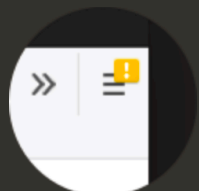

2. After you have enabled the browser extension from the top right menu, please refresh your browser from the link below.

[Refresh browser](#)

CLOSE